



# INSTALLATION AND SET UP FOR HOMEAWARE™ TRANSMITTER HA360VPT – Remote Phone Transmitter

The HomeAware™ HA360VPT Remote Phone Transmitter will alert you to a phone call, video relay service or a fax via the HomeAware™ Main unit.




## **Setting Up Your Transmitter**

The HomeAware™ HA360VPT Remote Phone Transmitter plugs into an RJ11 telephone jack (i.e. wall phone jack, pass-through phone jack on a phone or fax machine, or a phone jack provided by a VoIP or VRS hardware.)

### **Install Batteries**

The HomeAware™ Remote Phone Transmitter comes with two #357 silver oxide long lasting batteries that need to be installed into the bottom of the unit. Once installed the red light should light when the button is pushed. If the light does not show, you may have the batteries in backwards.

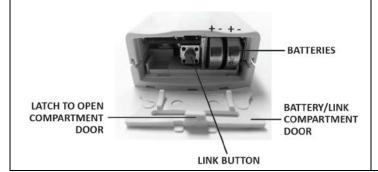

## **Link Button Set Up**

This procedure will link the HA360VPT – Remote Phone Transmitter to your Main unit. The Main unit will also provide an alert to all HomeAware™ remote receivers, and to original Sonic Alert receivers when using an HA360BU bridge unit.

## To link a transmitter to your HomeAware™ Main unit:

- Press Set Up button and the word LINK will be shown on the display, press Enter. LINK will begin blinking. (If LINK mode is entered by accident, simply press the BACK button to exit and return to the clock.)
- 2. Press and hold down the Link button on the bottom of your transmitter for about 1 second. The red LED on the transmitter will light up. Release the button and the words PICK A DECIVE will begin scrolling across the display. PHONE will then be shown on the display, press Enter if you want to be alerted to incoming phone call or press the up (Tone) button to switch to VRS or FAX, press Enter when the desired device is shown.
- The HomeAware<sup>™</sup> main unit will briefly show **DONE**, and return to the time display. Your transmitter is now connected and ready to test and use.
- 4. Incoming phone calls will now send a signal to the Main unit. You also can use the button on the front of the unit to send a test signal to the Main unit.







## INSTALLATION AND SET UP FOR HOMEAWARE™ TRANSMITTER **HA360VPT – Remote Phone Transmitter**

## **FCC Requirements**

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions; (1) This device might not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide responsible protect against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions manual, may cause harmful interference to radio communications. However, there is no guarantee that interference will not interfere in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- ii. Increase the separation between the equipment and receiver.
- iii. Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- iv. Consult the dealer or and experienced radio /TV technician for help

Modification not expressly approved by the manufacturer could void the user's authority to operate the equipment under FCC rules.

## **GENERAL DATA**

UL listed Power Adaptor: AC100-240V~, 50/60Hz

Power: Batteries 2 x SR44



The Lightening flash and arrowhead within the triangle is a warning sign alerting you of dangerous voltage inside the product.

## **CAUTION:**

TO REDUCE THE RISK OF ELECTRICAL SHOCK, DO NOT REMOVE THE COVER (OR BACK) FROM THE PRODUCT. NO USER SERVICEABLE **PARTS** INSIDE. REFER SERVICING TO YOUR SONIC

ALERT DEALER.



The exclamation mark within the triangle is a warning sign alerting you of important instructions accompanying the product.

WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS APPLIANCE TO RAIN OR MOISTURE.

The normal function of the product may be disturbed by Strong Electro Magnetic Interference. If so, simply reset the product to resume normal operation by following the instruction manual. In case the function could not resume, please use the product in other location.

Attention should be drawn to the environmental aspects of battery disposal.



This indicating separate collection for electrical and electronic equipment.

#### **SONIC ALERT'S 1 YEAR LIMITED WARRANTY**

The HomeAware™ HA360VPT are warranted against manufacturing defects in materials and workmanship for one (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the HomeAware™ HA360VPT without charge for parts and

Simply contact our customer service department to obtain an RA (return authorization number) and instructions on how to process your product for return. Call 1-888-864-2446 or e-mail to RMA@sonicalert.info. You'll need a copy of your receipt to process the return authorization.

**Sonic Alert Warranty Center** 1081 West Innovation Dr. Kearney, MO 64060, USA.



Made in China Sonic Alert, Troy, MI 48083 www.SonicAlert.com 1-888-864-244